

Disinfectants for System Surfaces

The following disinfectants can be used to disinfect all surfaces of the system (including probe holders) except for the monitor and touchscreen.

Locale	Product name	Active ingredient	Manufacturer	Type
EUROPE	Perasafe	Citric acid	LANXESS Deutschland GmbH	Powder
USA	Sani-Cloth® bleach Germicidal Disposable Wipe	Sodium Hypochlorite	PDI	Wipe
USA	T.Spray II	Quaternary Ammonium (N-Alkyl)	Pharmaceutical Innovations, Inc.	Spray
USA	PROTEX® ULTRA DISINFECTANT WIPES	Quaternary Ammonium (N-Alkyl)	Parker Laboratory Inc.	Wipe
EUROPE	Wip'Anios Excel wipes	Quaternary Ammonium (N-Alkyl)	Laboratoires Anios	Wipe
EUROPE	Surfa'safe	Quaternary Ammonium (N-Alkyl)	Laboratoires Anios	Liquid
KOREA	Clorox (Crorox 1 : water10)	Hypochlorous acid	Clorox	Liquid
KOREA	Enzysept PLUS Wipes	Quaternary Ammonium (N-Alkyl)	Wooil C&Tech	Wipe
KOREA	ED Wipes	Quaternary Ammonium (N-Alkyl)	MH healthcare	Wipe
KOREA	ANIOSURF Premium NPC	Quaternary Ammonium (N-Alkyl)	Laboratoires Anios	Liquid
EUROPE	Vikron S	Pentapotassium, Succinic acid	LANXESS	Powder
USA	Bleach Germicidal Wipes	Sodium Hypochlorite	The Chlorox Company	Wipe



CAUTION:

- ▶ Using an inappropriate disinfection procedure may damage the system. Make sure that you check the disinfectant's expiry date.
- ▶ Mix the disinfectant solution to the strength specified on the instruction of the disinfectant manufacturer.

Disinfectants for Monitor Surface

The following disinfectants can be used to disinfect the monitor and touchscreen.

Locale	Product name	Active ingredient	Manufacturer	Type
EUROPE	Perasafe	Citric acid	LANXESS Deutschland GmbH	Powder
USA	Sani-Cloth® bleach Germicidal Disposable Wipe	Sodium Hypochlorite	PDI	Wipe
USA	T.Spray II	Quaternary Ammonium (N-Alkyl)	Pharmaceutical Innovations, Inc.	Spray
USA	PROTEX® ULTRA DISINFECTANT WIPES	Quaternary Ammonium (N-Alkyl)	Parker Laboratory Inc.	Wipe
EUROPE	Wip'Anios Excel wipes	Quaternary Ammonium (N-Alkyl)	Laboratoires Anios	Wipe
KOREA	Clorox (Crorox 1 : water 10)	Hypochlorous acid	Clorox	Liquid
KOREA	Enzysept PLUS Wipes	Quaternary Ammonium (N-Alkyl)	Wooil C&Tech	Wipe
KOREA	ED Wipes	Quaternary Ammonium (N-Alkyl)	MH healthcare	Wipe
KOREA	ANIOSURF Premium NPC	Quaternary Ammonium (N-Alkyl)	Laboratoires Anios	Liquid
USA	Bleach Germicidal Wipes	Sodium Hypochlorite	The Chlorox Company	Wipe



CAUTION: When using a disinfectant, wipe it carefully with a soft cloth.


Using Ultrasound Gel

For successful acoustic signal transmission, please only use the ultrasound gels approved by Samsung Medison.



CAUTION:

- ▶ When applying the ultrasonic gel to the probe, make sure that the tip of the gel tube does not touch the surface of the probe lens.
- ▶ The tip of the gel tube may cause damage to the probe lens.

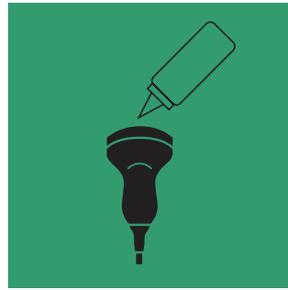

Cleaning, Disinfecting, and Sterilizing Probes

All probes must be cleaned and disinfected after use. Cleaning is an important procedure that must be carried out before disinfecting probes. For information about cleaning and disinfecting probes, please refer to the relevant chapter. Using an inappropriate disinfectant may damage the probe.



WARNING:

- ▶ Always use protective equipment such as face mask, eyewear, and gloves when cleaning, disinfecting, and sterilizing probes.
- ▶ Inspect the housing, strain relief, lens and seal for damage, and check for any functional degradation after cleaning and disinfecting the probe.
- ▶ Using an inappropriate cleaning or disinfecting agent may damage the probe.

Information regarding Detergent, Disinfectant, and Ultrasound Gel

Reprocessing Different Types of Probes

To maintain the performance of ultrasound probes, proper maintenance is required.

Ultrasound probes are classified into critical, semi-critical, and non-critical medical devices, in accordance with FDA guidance* and the Hygiene Requirements for the Reprocessing of Medical Devices set forth in the guidelines of the Robert Koch Institute (RKI) in Germany. Therefore, you should use the cleaning, disinfection, and sterilization methods appropriate for each classification.

Table – Selecting Appropriate Probe Care Methods

Classification Criteria	Contact Area	Application Probe	Level Selection
Non-critical medical devices	Intact skin	Curved, Linear, and Phased array probes	Low level disinfection
Semi-critical medical devices	Mucous membrane, damaged skin	Endocavity, TEE	High level disinfectant or disinfection
Critical medical devices	Blood, sterile tissue, etc.	Intraoperative	Sterilization

* Guidance for Industry and FDA Staff – Marketing Clearance of Diagnostic Ultrasound Systems and Transducers - Appendix E

* The FDA reprocessing guidance ‘Reprocessing Medical Devices in Health Care Settings: Validation Methods and Labeling, Guidance for Industry and Food and Drug Administration Staff’ March 17, 2015, (<https://www.fda.gov/media/80265/download>)

The care method for your probes determines the appropriate disinfectant for your probe. An appropriate detergent, disinfectant or ultrasound gel should be used for all probes. For details about compatible detergent, disinfectants, and ultrasound gel, please see ‘Disinfectants Matrix’ on Samsung Medison website and the User Guide.

- ▶ User Guide: This is provided as a booklet upon purchase of the product.

Disinfectants Matrix										EV3-10B	LV3-14A V6-12	C1-4EC	C51-4	SC1-6	CA1-7A CA1-7AD	C2-4/20	C2-4ES	C2-5	CA2-6BM	
No.	Classification	Name	Type	Disinfectant Level	Manufacture	Active Ingredient														
1	Cleaner	Alkazyme	Liquid	Cleaner	Alkapharm UK Ltd.	Proteolytic Enzymes														
2	Cleaner	Clearyme	Liquid	Cleaner	Advanced Sterilization Products	Proteolytic Enzymes														
3	Cleaner	Trophon® companion cleaning wipes	Wipe	Cleaner	Nanosonics Limited	Quaternary Ammonium(N-Alkyl)														
4	Cleaner	Enzol	Liquid	Cleaner	Johnson&Johnson	Dodecylphenolethoxylate, Sodium xylene Sulfonate														
5	Cleaner	Ethanol 75%	Liquid	Cleaner	All manufacture	Alcohol														
6	Cleaner	Isopropyl alcohol 70%	Liquid	Cleaner	All manufacture	Alcohol														
7	Cleaner	Isopropyl alcohol 80%	Liquid	Cleaner	All manufacture	Alcohol														
8	Cleaner	Klenzyme	Liquid	Cleaner	Steris	Proteolytic Enzymes														
9	Cleaner	McKesson	Liquid	Cleaner	McKesson Medical-Surgical Inc.	PCMI(Chloroxylenol)														
10	Cleaner	Meliorama	Liquid	Cleaner	Metrex	Propylene Glycol														
11	Cleaner	Aniozyme D01	Liquid	Cleaner	Laboratoires Anios	Proteolytic Enzymes														
12	Disinfectants	Protes disinfectant spray	Spray	Low	Parker Laboratories, Inc.	Quaternary Ammonium(N-Alkyl)														
13	Disinfectants	Anios Excel Surface Premium	Liquid	Low	Laboratoires Anios	Quaternary Ammonium(N-Alkyl)														
14	Disinfectants	Clinel Universal disinfecting wipes	Wipe	Low	GAMA Healthcare Ltd	Quaternary Ammonium(N-Alkyl)														
15	Disinfectants	Assept-Wipes	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)														
16	Disinfectants	Assept-Wipes II	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)														
17	Disinfectants	Bleach 5.25%	Liquid	Low	All manufacture	Sodium Hypochlorite														
18	Disinfectants	CarWipes	Wipe	Low	Metrex	Alcohol														
19	Disinfectants	Cleansept Wipe forte	Wipe	Low	Dr Schumacher GmbH	Quaternary Ammonium(N-Alkyl)														
20	Disinfectants	Cleansept Wipes	Wipe	Low	Dr Schumacher GmbH	Quaternary Ammonium(N-Alkyl)														
21	Disinfectants	Gigasept AF	Liquid	Low	Schulke & Mayr GmbH	Quaternary Ammonium(N-Alkyl)														
22	Disinfectants	Incidin Foam	Spray	Low	Ecolab	Quaternary Ammonium(N-Alkyl)														
23	Disinfectants	Melior Wipes	Wipe	Low	Metrex	Alcohol														
24	Disinfectants	Milton anti bacterial solution	Liquid	Low	Milton	Sodium Hypochlorite														
25	Disinfectants	Nuclean	Liquid	Low	Alkapharm	Alcohol														
26	Disinfectants	Sani-Cloth Active	Wipe	Low	PII	Quaternary Ammonium(N-Alkyl)														
27	Disinfectants	Sani-Cloth AF Germicidal	Wipe	Low	PII	Quaternary Ammonium(N-Alkyl)														
28	Disinfectants	Sani-Cloth Germicidal	Wipe	Low	PII	Quaternary Ammonium(N-Alkyl)														
29	Disinfectants	Sani-Cloth Bleach Germicidal	Wipe	Low	PII	Sodium Hypochlorite														
30	Cleaner	Endozime	Liquid	Cleaner	The Ruhof Corporation	Proteolytic Enzymes														
31	Disinfectants	Sani-Cloth HB	Wipe	Low	PII	Quaternary Ammonium(N-Alkyl)														
32	Disinfectants	Sani-Cloth Plus	Wipe	Low	PII	Quaternary Ammonium(N-Alkyl)														
33	Disinfectants	Sani-Cloth Plus	Wipe	Low	PII	Quaternary Ammonium(N-Alkyl)														
34	Disinfectants	Sonowipes	Wipe	Low	Advanced Ultrasound Solutions Inc.	Quaternary Ammonium(N-Alkyl)														
35	Disinfectants	Ster-Bar Bio	Liquid	Low	Ecolab	Quaternary Ammonium(N-Alkyl)														
36	Disinfectants	Super Sani-Cloth	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)														
37	Cleaner	Transsept spray	Spray	Cleaner	Parker Laboratories, Inc.	Alcohol														
38	Disinfectants	T-Spray	Spray	Low	Pharmaceutical Innovations, Inc.	Quaternary Ammonium(N-Alkyl)														
39	Disinfectants	T-SprayII	Spray	Low	Pharmaceutical Innovations, Inc.	Quaternary Ammonium(N-Alkyl)														
40	Disinfectants	Aniozyme 1000	Liquid	High	Laboratoires Anios	Peracetic Acid														
41	Disinfectants	Cidex	Liquid	High	Advanced Sterilization Products	Glutaraldehyde														
42	Disinfectants	Cidex OPA	Liquid	High	Advanced Sterilization Products	Ortho-phthalaldehyde														
43	Disinfectants	Cidex Plus	Liquid	High	Advanced Sterilization Products	Glutaraldehyde														
44	Disinfectants	Gigasept	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde, formaldehyde														
45	Disinfectants	Gigasept FF	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde														
46	Disinfectants	Hibiclens	Liquid	High	Regis Medical	Chlorhexidine gluconate														
47	Disinfectants	Metricide	Liquid	High	Metrex	Glutaraldehyde														
48	Disinfectants	Metricide 28	Liquid	High	Metrex	Glutaraldehyde														
49	Disinfectants	Metricide 30	Liquid	High	Metrex	Glutaraldehyde														
50	Disinfectants	Metricide OPA Plus	Liquid	High	Metrex	Ortho-phthalaldehyde														
51	Disinfectants	Omnicide - FQ2	Liquid	High	Coventry Chemicals Limited	Glutaraldehyde														
52	Disinfectants	Omnicide 14	Liquid	High	METREX RESEARCH CORPORATION	Glutaraldehyde														
53	Disinfectants	Omnicide 28	Liquid	High	METREX RESEARCH CORPORATION	Glutaraldehyde														
54	Disinfectants/Sterilizer	PerASafe™	Powder	High	Dupont	Citric Acid														
55	Disinfectants	Salvanios pH 7	Liquid	High	Laboratoires Anios	Glutaraldehyde														
56	Disinfectants	Salvanios pH10	Liquid	High	Laboratoires Anios	Glutaraldehyde														
57	Disinfectants	Sekusept Extra	Liquid	High	Ecolab	Glutaraldehyde														
58	Disinfectants	Sekusept Plus	Liquid	Low	Ecolab	Glucopyranin														
59	Disinfectants	Sporox	Liquid	High	DSHealthcare Inc.	Hydrogen Peroxide														
60	Disinfectants	Sporox II	Liquid	High	DSHealthcare Inc.	Hydrogen Peroxide														
61	Disinfectants	Steranios 2%	Liquid	High	Laboratoires Anios	Glutaraldehyde														
62	Disinfectants	REVITALiDx RESERT	Liquid	High	Steris	Hydrogen Peroxide														
63	Disinfectants	Surfaces Hautes	Spray	High	Laboratoires Anios	Glutaraldehyde														
64	Disinfectants	Tristel Duo	Foam	High	Tristel Solutions Limited	Chlorine dioxide														
65	Disinfectants	Tristel Sporticial Wipe	Wipe	High	Tristel Solutions Limited	Chlorine dioxide														
66	Disinfectants	Virkon	Powder	High	Dupont	Potassium peroxymonosulfate														
67	Disinfectants	Wavecide-01	Liquid	High	National Therapy Products Inc.	Hydrogen Peroxide														
68	Disinfectants	Gigasept PAA	Liquid	High	Schulke & Mayr GmbH	Hydrogen peroxide, Acetic Acid, Peracetic Acid														
69	Disinfection System	Antibacna 51	Liquid	High	Germetics	UV														
70	Disinfection System	Trophon	Vapor	High	Nanosonics Limited	Hydrogen Peroxide														
71	Disinfection System	Trophon2	Vapor	High	Nanosonics Limited	Hydrogen Peroxide														
72	Gel	Aquasonics 100	Gel	Gel	Parker Laboratories, Inc.	NA														
73	Gel	Clear Image	Gel	Gel	NEC Medical Products	NA														
74	Gel	Natural Image	Gel	Gel	Sonotech Inc.	NA														
75	Gel	Scan	Gel	Gel	Parker Laboratories, Inc.	NA														
76	Gel	Sonogel	Gel	Gel	Sonogel Vertriebs GmbH	NA														
77	Gel	WaveLength	Gel	Gel	National Therapy Products Inc.	NA														
78	Disinfectants	Acrylan	Liquid	Low	Antiseptica Chem. Phar. Produkte	Quaternary Ammonium(N-Alkyl)														
79	Disinfectants	SEPTOL Linettes	Wipe	Low	ACTEON	Quaternary Ammonium(N-Alkyl)														
80	Disinfectants	Wip Anios Excel	Wipe	Low	Laboratoires Anios	Quaternary Ammonium(N-Alkyl)														
81	Cleaner	Empower	Liquid	Cleaner	Metrex	Propylene glycol														
82	Disinfectants	Hexanios GR	Liquid	Low	Laboratoires Anios	Quaternary Ammonium(N-Alkyl)														
83	Disinfectants	Oxivir TB Wipe	Wipe	High	Diversey	Hydrogen Peroxide														
84	Disinfectants	Bacilcol 30 Foam	Spray	Low	BODE Chemie GmbH (HARTMANN)	Quaternary Ammonium(N-Alkyl), Alcohol														
85	Disinfectants	Carvicide	Spray	Low	Metrex	Quaternary Ammonium(N-Alkyl), Alcohol														
86	Disinfectants	Clinel Universal Spray	Spray	Low	GAMA Healthcare Ltd	Quaternary Ammonium(N-Alkyl)														
87	Disinfectants	Gigasept FF neu	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde														
88	Disinfectants	Envirocide	Spray	Low	Metrex	Quaternary Ammonium(N-Alkyl), Alcohol														
89	Disinfectants	MetricGuard	Spray	Low	Metrex	Quaternary Ammonium(N-Alkyl), Alcohol														
90	Disinfectants	Chlorox 100PPM	Liquid	High	All manufacture	Chlorine compound														
91	Disinfectants	Clorox DF	Liquid	High	MDG COMPANY GmbH	Sodium percarbonate														
92	Disinfectants	SCOPANIOS RC	Liquid	High	Laboratoires Anios	Hydrogen peroxide, Peracetic Acid														
93	Cleaner	Empower Foam	Liquid	Cleaner	Metrex	Polyethylene glycol														
94	Cleaner	MetricLube	Liquid	Cleaner	Metrex	Propylene glycol														
95	Cleaner	MetricShine	Liquid	Cleaner	Metrex	Phosphoric Acid, Glycolic Acid														
96	Cleaner	MetricSponge	Liquid	Cleaner	Metrex	Propylene glycol														
97	Sterilization system	V-PRO	Vapor	High	Steris	Hydrogen peroxide														
98	Disinfectants	Chloroxidox Lesung	Liquid	High	vitalundfimt100 UG	Chlorine dioxide														
99	Cleaner	Endozime Xtreme Power	Liquid	Cleaner	Ruhof Healthcare	Proteolytic Enzymes														
100	Disinfectants	Surfactos Premium	Liquid	Low	Anios	Quaternary Ammonium(N-Alkyl), Alcohol														
101	Disinfectants	Virkon 5	Liquid	High	Dupont	Potassium peroxymonosulfate														
102	Disinfectants	V-Wipes	Wipe	Low	Whiteley Corporation	Alcohol														
103	Disinfectants	terralin PAA																		

No.	Classification	Name	Type	Disinfectant Matrix			Convex													
				Disinfectant Level	Manufacture	Active Ingredient	C2-6 C2-6IC	CN2-8	C2-8 C2-8ED	CA2-8A CA2-8AD CF2-8	CA2-9A CA2-9AD	C3-7EP	CA3-10A	CA4-9/10ED	CN4-9	C5-8 CF4-9				
1	Cleaner	Alkazyne	Liquid	Cleaner	Alkapharm UK Ltd.	Proteolytic Enzymes														
2	Cleaner	Clayzyme	Liquid	Cleaner	Advanced Sterilization Products	Proteolytic Enzymes														
3	Cleaner	Trophon® companion cleaning wipes	Wipe	Cleaner	Nanosonics Limited	Quaternary Ammonium(N-Alkyl)														
4	Cleaner	Enzol	Liquid	Cleaner	Johnson&Johnson	Dodecylphenolethoxylate, Sodium xylene Sulfonate														
5	Cleaner	Ethanol 75%	Liquid	Cleaner	All manufacture	Alcohol														
6	Cleaner	Isopropyl alcohol 70%	Liquid	Cleaner	All manufacture	Alcohol														
7	Cleaner	Isopropyl alcohol 80%	Liquid	Cleaner	All manufacture	Alcohol														
8	Cleaner	Klenzyme	Liquid	Cleaner	Steris	Proteolytic Enzymes														
9	Cleaner	McKesson	Liquid	Cleaner	McKesson Medical-Surgical Inc.	PCMI(Chloroxylenol)														
10	Cleaner	Micromax	Liquid	Cleaner	Metrex	Propylene Glycol														
11	Cleaner	Aniozyme D01	Liquid	Cleaner	Proteolytic Enzymes	Quaternary Ammonium(N-Alkyl)														
12	Disinfectants	Prolex disinfectant spray	Spray	Low	Parker Laboratories, Inc.	Quaternary Ammonium(N-Alkyl)														
13	Disinfectants	Anios Excel Surface Premium	Liquid	Low	Laboratories Anios	Quaternary Ammonium(N-Alkyl)														
14	Disinfectants	Clinel® Universal Antiseptic Wipes	Wipe	Low	GAMA Healthcare Ltd	Quaternary Ammonium(N-Alkyl)														
15	Disinfectants	Assepti-Wipes	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)														
16	Disinfectants	Assepti-Wipes II	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)														
17	Disinfectants	Bleachch 5.25%	Liquid	Low	All manufacture	Sodium Hypochlorite														
18	Disinfectants	CarWipes	Wipe	Low	Metrex	Alcohol														
19	Disinfectants	Cleansept Wipe forte	Wipe	Low	Dr Schumacher GmbH	Quaternary Ammonium(N-Alkyl)														
20	Disinfectants	Cleansept Wipes	Wipe	Low	Dr Schumacher GmbH	Quaternary Ammonium(N-Alkyl)														
21	Disinfectants	Gigasept AF	Liquid	Low	Schulke & Mayr GmbH	Quaternary Ammonium(N-Alkyl)														
22	Disinfectants	Incidin Foam	Spray	Low	Ecolab	Quaternary Ammonium(N-Alkyl)														
23	Disinfectants	Meltris Wipes	Wipe	Low	Metrex	Alcohol														
24	Disinfectants	Milton anti bacterial solution	Liquid	Low	Milton	Sodium Hypochlorite														
25	Disinfectants	Nuclean	Liquid	Low	Alkapharm	Alcohol														
26	Disinfectants	Sani-Cloth Active	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)														
27	Disinfectants	Sani-Cloth AF-Germicidal	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)														
28	Disinfectants	Sani-Cloth AF-Germicidal	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)														
29	Disinfectants	Sani-Cloth Bleach Germicidal	Wipe	Low	PDI	Sodium Hypochlorite														
30	Cleaner	Endozime	Liquid	Cleaner	The Ruhoff Corporation	Proteolytic Enzymes														
31	Disinfectants	Sani-Cloth HB	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)														
32	Disinfectants	Sani-Cloth Plus	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)														
33	Disinfectants	Sani-Cloth Plus	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)														
34	Disinfectants	Sonowipes	Wipe	Low	Advanced Ultrasound Solutions Inc.	Quaternary Ammonium(N-Alkyl)														
35	Disinfectants	Ster-Bar Cloth	Liquid	Low	Ecolab	Quaternary Ammonium(N-Alkyl)														
36	Disinfectants	Super Sani-cloth	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)														
37	Cleaner	Transsept spray	Spray	Cleaner	Parker Laboratories, Inc.	Alcohol														
38	Disinfectants	T-Spray	Spray	Low	Pharmaceutical Innovations, Inc.	Quaternary Ammonium(N-Alkyl)														
39	Disinfectants	T-Spray	Spray	Low	Pharmaceutical Innovations, Inc.	Quaternary Ammonium(N-Alkyl)														
40	Disinfectants	Aniozyme 1000	Liquid	High	Laboratories Anios	Peracetic Acid														
41	Disinfectants	Cidex	Liquid	High	Advanced Sterilization Products	Glutaraldehyde														
42	Disinfectants	Cidex OPA	Liquid	High	Advanced Sterilization Products	Ortho-phthalaldehyde														
43	Disinfectants	Cidex Plus	Liquid	High	Advanced Sterilization Products	Glutaraldehyde														
44	Disinfectants	Gigasept	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde, formaldehyde														
45	Disinfectants	Gigasept FF	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde														
46	Disinfectants	Hibiclens	Liquid	High	Regis Medical	Chlorhexidine gluconate														
47	Disinfectants	Metricide	Liquid	High	Metrex	Glutaraldehyde														
48	Disinfectants	Metricide 28	Liquid	High	Metrex	Glutaraldehyde														
49	Disinfectants	Metricide 30	Liquid	High	Metrex	Glutaraldehyde														
50	Disinfectants	Metricide OPA Plus	Liquid	High	Metrex	Ortho-phthalaldehyde														
51	Disinfectants	Omnicide - FC2	Liquid	High	Coventry Chemicals Limited	Glutaraldehyde														
52	Disinfectants	Omnicide 14	Liquid	High	METREX RESEARCH CORPORATION	Glutaraldehyde														
53	Disinfectants	Omnicide 28	Liquid	High	METREX RESEARCH CORPORATION	Glutaraldehyde														
54	Disinfectants/Sterilizer	PerASafe™	Powder	High	Dupont	Citric Acid														
55	Disinfectants	Salvanios pH 7	Liquid	High	Laboratories Anios	Glutaraldehyde														
56	Disinfectants	Salvanios pH10	Liquid	High	Laboratories Anios	Glutaraldehyde														
57	Disinfectants	Sekusept Extra	Liquid	High	Ecolab	Glutaraldehyde														
58	Disinfectants	Sekusept Plus	Liquid	Low	Ecolab	Glucoprotamin														
59	Disinfectants	Sporon	Liquid	High	DSHealthcare Inc.	Hydrogen Peroxide														
60	Disinfectants	Sporon II	Liquid	High	DSHealthcare Inc.	Hydrogen Peroxide														
61	Disinfectants	Steranios 2%	Liquid	High	Laboratories Anios	Glutaraldehyde														
62	Disinfectants	REVITALiDx RESERT	Liquid	High	Steris	Hydrogen Peroxide														
63	Disinfectants	Surfaces Hautes	Spray	High	Laboratories Anios	Glutaraldehyde														
64	Disinfectants	Tristel Duo	Foam	High	Tristel Solutions Limited	Chlorine dioxide														
65	Disinfectants	Tristel Sporticial Wipe	Wipe	High	Tristel Solutions Limited	Chlorine dioxide														
66	Disinfectants	Virkon	Powder	High	Dupont	Potassium peroxymonosulfate														
67	Disinfectants	Wavicide-01	Liquid	High	National Therapy Products Inc.	Glutaraldehyde														
68	Disinfectants	Gigasept PAA	Liquid	High	Schulke & Mayr GmbH	Hydrogen peroxide, Acetic Acid, Peracetic Acid														
69	Disinfection System	Antiseptax S1	Liquid	High	germies.	UV-C														
70	Disinfection System	Trophon	Vapor	High	Nanosonics Limited	Hydrogen Peroxide														
71	Disinfection System	Trophon2	Vapor	High	Nanosonics Limited	Hydrogen Peroxide														
72	Gel	Aquazions 100	Gel	Gel	Parker Laboratories, Inc.	NA														
73	Gel	Clear Image	Gel	Gel	NET Medical Products	NA														
74	Gel	Natural Image	Gel	Gel	Sonotech Inc.	NA														
75	Gel	Scan	Gel	Gel	Parker Laboratories, Inc.	NA														
76	Gel	Sonogel	Gel	Gel	Sonogel Vertriebs GmbH	NA														
77	Gel	Wavelength	Gel	Gel	National Therapy Products Inc.	NA														
78	Disinfectants	Acrylan	Liquid	Low	Antiseptica Chem. Fabr. Produkte	Quaternary Ammonium(N-Alkyl)														
79	Disinfectants	SEPTOL Linettes	Wipe	Low	ACTEON	Quaternary Ammonium(N-Alkyl)														
80	Disinfectants	Wip Anios Excel	Wipe	Low	Laboratories Anios	Quaternary Ammonium(N-Alkyl)														
81	Cleaner	EmpPower	Liquid	Cleaner	Metrex	Propylene glycol														
82	Disinfectants	Hexanios GR	Liquid	Low	Laboratories Anios	Quaternary Ammonium(N-Alkyl)														
83	Disinfectants	Oxivir TB Wipe	Wipe	High	Diversey	Hydrogen Peroxide														
84	Disinfectants	Bacilcol 30 Foam	Spray	Low	BODE Chemie GmbH (HARTMANN)	Quaternary Ammonium(N-Alkyl), Alcohol														
85	Disinfectants	Carvicide	Spray	Low	Metrex	Quaternary Ammonium(N-Alkyl), Alcohol														
86	Disinfectants	Clinel Universal Spray	Liquid	Low	GAMA Healthcare Ltd	Quaternary Ammonium(N-Alkyl)														
87	Disinfectants	Gigasept FF neu	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde														
88	Disinfectants	Envirocide	Spray	Low	Metrex	Quaternary Ammonium(N-Alkyl), Alcohol														
89	Disinfectants	MetriGuard	Spray	Low	Metrex	Quaternary Ammonium(N-Alkyl), Alcohol														
90	Disinfectants	Chlorine 100PPM	Liquid	High	All manufacture	Chlorine compound														
91	Disinfectants	Clorox NF	Liquid	High	MDI COMPANY GmbH	Sodium percarbonate														
92	Disinfectants	SCOPANIOS RC	Liquid	High	Laboratories Anios	Hydrogen peroxide, Peracetic Acid														
93	Cleaner	EmpPower Foam	Liquid	Cleaner	Metrex	Polyethylene glycol														
94	Cleaner	MetriLube	Liquid	Cleaner	Metrex	Propylene glycol														
95	Cleaner	MetriShine	Liquid	Cleaner	Metrex	Phosphoric Acid, glycolic Acid														
96	Cleaner	MetriSpray	Liquid	Cleaner	Metrex	Propylene glycol														
97	Sterilization system	V-PRO	Vapor	High	Steris	Hydrogen peroxide														
98	Disinfectants	Chlorodox Lösung	Liquid	High	vitalundformit100 UG	Chlorine dioxide														
99	Cleaner	Endozime Xtreme Power	Liquid	Cleaner	Ruhoff Healthcare	Proteolytic Enzymes														
100	Surfactants	Surfactax Premium	Liquid	Low	Anios	Quaternary Ammonium(N-Alkyl), Alcohol														
101	Disinfectants	Virkon S	Liquid	High	Dupont	Potassium peroxymonosulfate														
102	Disinfectants	V-Wipes	Wipe	Low	Whiteley Corporation	Alcohol														
103	Disinfectants	terralin PAA	Liquid	High	Schulke	Hydro														

Disinfectants Matrix							Endocavity									
No.	Classification	Name	Type	Disinfectant Level	Manufacture	Active Ingredient	ECA-9	ECA-9IS	ERA-9/10ED	EVA-9/10ED	EVNA-9	VR5-9	EA2-11B	EA2-11AR EA2-11AV	ES-12A	LA2-9A L4-7
1	Cleaner	Alkazyne	Liquid	Cleaner	Alkapharm UK Ltd.	Proteolytic Enzymes										
2	Cleaner	Clayzyme	Liquid	Cleaner	Advanced Sterilization Products	Proteolytic Enzymes										
3	Cleaner	Trophon® companion cleaning wipes	Wipe	Cleaner	Nanosonics Limited	Quaternary Ammonium(N-Alkyl)										
4	Cleaner	Enzol	Liquid	Cleaner	Johnson&Johnson	Dodecylphenolethoxylate, Sodium xylene Sulfonate										
5	Cleaner	Ethanol 75%	Liquid	Cleaner	All manufacture	Alcohol										
6	Cleaner	Isopropyl alcohol 70%	Liquid	Cleaner	All manufacture	Alcohol										
7	Cleaner	Isopropyl alcohol 80%	Liquid	Cleaner	All manufacture	Alcohol										
8	Cleaner	Klenzyme	Liquid	Cleaner	Steris	Proteolytic Enzymes										
9	Cleaner	McKesson	Liquid	Cleaner	McKesson Medical-Surgical Inc.	PCMI(Chloroxylenol)										
10	Cleaner	Melrose	Liquid	Cleaner	Metrex	Propylene Glycol										
11	Cleaner	Aniozyme D01	Liquid	Cleaner	Laboratories Anios	Proteolytic Enzymes										
12	Disinfectants	Prolex disinfectant spray	Spray	Low	Parker laboratories, Inc.	Quaternary Ammonium(N-Alkyl)										
13	Disinfectants	Anios Excel Surface Premium	Liquid	Low	Laboratories Anios	Quaternary Ammonium(N-Alkyl)										
14	Disinfectants	Clinel Universal	Wipe	Low	GAMA Healthcare Ltd	Quaternary Ammonium(N-Alkyl)										
15	Disinfectants	Assepti-Wipes	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)										
16	Disinfectants	Assepti-Wipes II	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)										
17	Disinfectants	Bleach 5.25%	Liquid	Low	All manufacture	Sodium Hypochlorite										
18	Disinfectants	CarWipes	Wipe	Low	Metrex	Alcohol										
19	Disinfectants	Cleansept Wipe forte	Wipe	Low	Dr Schumacher GmbH	Quaternary Ammonium(N-Alkyl)										
20	Disinfectants	Cleansept Wipes	Wipe	Low	Dr Schumacher GmbH	Quaternary Ammonium(N-Alkyl)										
21	Disinfectants	Gigasept AF	Liquid	Low	Schulke & Mayr GmbH	Quaternary Ammonium(N-Alkyl)										(30)
22	Disinfectants	Incidin Foam	Spray	Low	Ecolab	Quaternary Ammonium(N-Alkyl)										
23	Disinfectants	Melrose Wipes	Wipe	Low	Metrex	Alcohol										
24	Disinfectants	Milton anti bacterial solution	Liquid	Low	Milton	Sodium Hypochlorite										
25	Disinfectants	Nuclean	Liquid	Low	Alkapharm	Alcohol										
26	Disinfectants	Sani-Cloth Active	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)										
27	Disinfectants	Sani-Cloth AF-Germicidal	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)										
28	Disinfectants	Sani-Cloth AF-Germicidal	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)										
29	Disinfectants	Sani-Cloth Bleach Germicidal	Wipe	Low	PDI	Sodium Hypochlorite										
30	Cleaner	Endozime	Liquid	Cleaner	The Ruhoff Corporation	Proteolytic Enzymes										
31	Disinfectants	Sani-Cloth HB	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)										(30)
32	Disinfectants	Sani-Cloth Plus	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)										(30)
33	Disinfectants	Sani-Cloth Plus	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)										(30)
34	Disinfectants	Sonowipes	Wipe	Low	Advanced Ultrasound Solutions Inc.	Quaternary Ammonium(N-Alkyl)										(50)
35	Disinfectants	Ster-Bar Blu	Liquid	Low	Ecolab	Quaternary Ammonium(N-Alkyl)										(50)
36	Disinfectants	Super Sani-cloth	Wipe	Low	PDI	Quaternary Ammonium(N-Alkyl)										(50)
37	Cleaner	Transi-spray	Spray	Cleaner	Parker Laboratories, Inc.	Alcohol										(15)
38	Disinfectants	T-Spray	Spray	Low	Pharmaceutical Innovations, Inc.	Quaternary Ammonium(N-Alkyl)										(20)
39	Disinfectants	T-SprayII	Spray	Low	Pharmaceutical Innovations, Inc.	Quaternary Ammonium(N-Alkyl)										(20)
40	Disinfectants	Aniozyme 1000	Liquid	High	Laboratories Anios	Peracetic Acid										x
41	Disinfectants	Cidex	Liquid	High	Advanced Sterilization Products	Glutaraldehyde										(50)
42	Disinfectants	Cidex OPA	Liquid	High	Advanced Sterilization Products	Ortho-phthalaldehyde										(15)
43	Disinfectants	Cidex Plus	Liquid	High	Advanced Sterilization Products	Glutaraldehyde										(25)
44	Disinfectants	Gigasept	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde, formaldehyde										(50)
45	Disinfectants	Gigasept FF	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde										(30)
46	Disinfectants	Hibiclens	Liquid	High	Regis Medical	Chlorhexidine gluconate										(50)
47	Disinfectants	Metricide	Liquid	High	Metrex	Glutaraldehyde										(50)
48	Disinfectants	Metricide 28	Liquid	High	Metrex	Glutaraldehyde										
49	Disinfectants	Metricide 30	Liquid	High	Metrex	Glutaraldehyde										
50	Disinfectants	Metricide OPA Plus	Liquid	High	Metrex	Ortho-phthalaldehyde										(50)
51	Disinfectants	Omicide-FC2	Liquid	High	Coventry Chemicals Limited	Glutaraldehyde										
52	Disinfectants	Omicide-14	Liquid	High	METREX RESEARCH CORPORATION	Glutaraldehyde										
53	Disinfectants	Omicide28	Liquid	High	METREX RESEARCH CORPORATION	Glutaraldehyde										
54	Disinfectants/Sterilizer	PeriSani®	Powder	High	Dupont	Citric Acid										(15)
55	Disinfectants	Salvanios pH 7	Liquid	High	Laboratories Anios	Glutaraldehyde										
56	Disinfectants	Salvanios pH10	Liquid	High	Laboratories Anios	Glutaraldehyde										
57	Disinfectants	Sekusept Extra	Liquid	High	Ecolab	Glutaraldehyde										
58	Disinfectants	Sekusept Plus	Liquid	Low	Ecolab	Glucoprotamin										
59	Disinfectants	Sporon	Liquid	High	DSHealthcare Inc.	Hydrogen Peroxide										x
60	Disinfectants	Sporon II	Liquid	High	DSHealthcare Inc.	Hydrogen Peroxide										
61	Disinfectants	Steranios 2%	Liquid	High	Laboratories Anios	Glutaraldehyde										
62	Disinfectants	REVITALiDx RESERT	Liquid	High	Steris	Hydrogen Peroxide										
63	Disinfectants	Surfaces Hautes	Spray	High	Laboratories Anios	Glutaraldehyde										
64	Disinfectants	Tristel Duo	Foam	High	Tristel Solutions Limited	Chlorine dioxide										
65	Disinfectants	Tristel Sportoidal Wipe	Wipe	High	Tristel Solutions Limited	Chlorine dioxide										
66	Disinfectants	Virkon	Powder	High	Dupont	Potassium peroxymonosulfate										
67	Disinfectants	Wavicide-01	Liquid	High	National Therapy Products Inc.	Glutaraldehyde										(50)
68	Disinfectants	Gigasept PAA	Liquid	High	Schulke & Mayr GmbH	Hydrogen peroxide, Acetic Acid, Peracetic Acid										(50)
69	Disinfection System	Ambaqua S1	Liquid	High	UV germics	UV										
70	Disinfection System	Trophon	Vapor	High	Nanosonics Limited	Hydrogen Peroxide										
71	Disinfection System	Trophon2	Vapor	High	Nanosonics Limited	Hydrogen Peroxide										
72	Gel	Aquazones 100	Gel	Gel	Parker Laboratories, Inc.	NA										
73	Gel	Clear Image	Gel	Gel	NEC Medical Products	NA										
74	Gel	Natural Image	Gel	Gel	Sonotech Inc.	NA										
75	Gel	Scan	Gel	Gel	Parker laboratories, Inc.	NA										
76	Gel	Sonogel	Gel	Gel	Sonogel Vertriebs GmbH	NA										
77	Gel	Wavelength	Gel	Gel	National Therapy Products Inc.	NA										
78	Disinfectants	Acrylan	Liquid	Low	Antiseptica Chem. Pharm. Produkte	Quaternary Ammonium(N-Alkyl)										
79	Disinfectants	SEPTOL Linettes	Wipe	Low	ACTEON	Quaternary Ammonium(N-Alkyl)										
80	Disinfectants	Wip Anios Excel	Wipe	Low	Laboratories Anios	Quaternary Ammonium(N-Alkyl)										
81	Cleaner	EmpPower	Liquid	Cleaner	Metrex	Propylene glycol										
82	Disinfectants	Hexanios GR	Liquid	Low	Laboratories Anios	Quaternary Ammonium(N-Alkyl)										
83	Disinfectants	Oxivir TB Wipe	Wipe	High	Diversey	Hydrogen Peroxide										
84	Disinfectants	Bacilcol 30 Foam	Spray	Low	BODE Chemie GmbH (HARTMANN)	Quaternary Ammonium(N-Alkyl), Alcohol										
85	Disinfectants	Carvicide	Spray	Low	Metrex	Quaternary Ammonium(N-Alkyl), Alcohol										
86	Disinfectants	Clinel Universal Spray	Liquid	Low	GAMA Healthcare Ltd	Quaternary Ammonium(N-Alkyl)										
87	Disinfectants	Gigasept FF neu	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde										
88	Disinfectants	Envirocide	Spray	Low	Metrex	Quaternary Ammonium(N-Alkyl), Alcohol										
89	Disinfectants	Metricidard	Spray	Low	Metrex	Quaternary Ammonium(N-Alkyl), Alcohol										
90	Disinfectants	Chlorox 100PPM	Liquid	High	All manufacture	Chlorine compound										
91	Disinfectants	Clorox NF	Liquid	High	MDi COMPANY GmbH	Sodium percarbonate										
92	Disinfectants	SCOPANIOS RC	Liquid	High	Laboratories Anios	Hydrogen peroxide, Peracetic Acid										
93	Cleaner	EmpPower Foam	Liquid	Cleaner	Metrex	Polyethylene glycol										
94	Cleaner	MetricLube	Liquid	Cleaner	Metrex	Propylene glycol										
95	Cleaner	MetricShine	Liquid	Cleaner	Metrex	Phosphoric Acid, glycolic Acid										
96	Cleaner	MetricSpray	Liquid	Cleaner	Metrex	Propylene glycol										
97	Sterilization system	V-PRO	Vapor	High	Steris	Hydrogen peroxide										
98	Disinfectants	Chloroxdiox Lesung	Liquid	High	vitalundformit100 UG	Chlorine dioxide										
99	Cleaner	Endozime Xtreme Power	Liquid	Cleaner	Ruhoff Healthcare	Proteolytic Enzymes										
100	Disinfectants	Surfactos Premium	Liquid	Low	Anios	Quaternary Ammonium(N-Alkyl), Alcohol										
101	Disinfectants	Virkon S	Liquid	High	Dupont	Potassium peroxymonosulfate										
102	Disinfectants	V-Wipes	Wipe	Low	Whiteley Corporation	Alcohol										
103	Disinfectants	terralin PAA	Liquid	High	Schulke	Hydrogen peroxide, Acetic Acid, Peracetic Acid										
104	Disinfectants	Bicide SPA/28	Liquid	High	Cantel Medical	Ortho-phthalaldehyde										
105	Cleaner	Tristel Pre-clean wipe	Wipe	Cleaner	Tristel Solutions Limited	Alcohol										
106	Disinfectants	Binquid S	Liquid	Low	Antiseptica Chem	Quaternary Ammonium(N-Alkyl), Alcohol										
107	Disinfectants	deconox 11 UNIVERSAL	Liquid	High	Borer Chemie AG	Potassium Pyrophosphate, Potassium hydroxide,										
108	Disinfectants	MEDI CLEN	Liquid	Low	UBM Cleaning Solution INC	Sodium hypochlorite										
109	Disinfectants	saFacide	Wipe	Low	Liebecos	Quaternary Ammonium(N-Alkyl)										
110	Disinfectants	SURFA SAFE	Liquid	Low	Anios	Quaternary Ammonium(N-Alkyl)										
111	Disinfectants	Mikrobaq Tissues	Wipe	Low	BODE Chemie GmbH	Quaternary Ammonium(N-Alkyl)										
112	Disinfectants	Meliseptol Wipes Sensitive	Wipe	Low	B. Braun Medical AG	Quaternary Ammonium(N-Alkyl)										
113	Disinfectants	Mikrocid sensitive liquid	Liquid	Low	Schulke	Quaternary Ammonium(N-Alkyl)										
114	Disinfectants	Mikrocid sensitive Wipes	Wipe	Low	Schulke	Quaternary Ammonium(N-Alkyl)										
115	Disinfectants	Mikrocid universal liquid	Liquid	Low	Schulke	Alcohol										
116	Disinfectants	Mikrocid universal Wipes premium	Wipe	Low	Schulke	Alcohol										
117	Disinfectants	Protex ultra	Wipe	Low	Parker laboratories, Inc.	Quaternary Ammonium(N-Alkyl)										
118	Disinfectants	Tristel Sportoidal Wipes	Wipe	Low	Tristel Solutions Limited	Quaternary Ammonium(N-Alkyl)										

Disinfectants Matrix							CW				TEE				
No.	Classification	Name	Type	Disinfectant Level	Manufacture	Active Ingredient	SP3-8	PA3-8B P3-8CA	PA4-12B PA-12	DP2B	CW2.0	CW4.0	CW6.0	DP8B	MMP3-7 MPT3-7
1	Cleaner	Alkazyme	Liquid	Cleaner	Alkapharm UK Ltd.	Proteolytic Enzymes	--	--	--	--	--	--	--	--	--
2	Cleaner	Cidezyme	Liquid	Cleaner	Advanced Sterilization Products	Proteolytic Enzymes	--	--	--	--	--	--	--	--	--
3	Cleaner	Trophon® companion cleaning wipes	Wipe	Cleaner	Nanosonics Limited	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
4	Cleaner	Enzol	Liquid	Cleaner	Johnson&Johnson	Dodecylphenylethoxylate, Sodium xylene Sulfonate	--	●	●	--	--	--	--	●	●
5	Cleaner	Ethanol 75%	Liquid	Cleaner	All manufacture	Alcohol	--	--	--	--	--	--	--	--	--
6	Cleaner	Isopropyl alcohol 70%	Liquid	Cleaner	All manufacture	Alcohol	●	●	●	--	--	--	--	--	●
7	Cleaner	Isopropyl alcohol 80%	Liquid	Cleaner	All manufacture	Alcohol	--	--	--	--	--	--	--	--	--
8	Cleaner	Klenzyme	Liquid	Cleaner	Steris	Proteolytic Enzymes	--	●	●	--	--	--	--	●	--
9	Cleaner	McKesson	Liquid	Cleaner	McKesson Medical-Surgical Inc.	PCMI(Chloroxylenol)	--	●	●	●	--	--	--	●	--
10	Cleaner	Metroxyme	Liquid	Cleaner	Metrox	Propylene Glycol	●	●	●	--	--	--	--	●	--
11	Cleaner	Aniozyme D01	Liquid	Cleaner	Laboratories Anios	Proteolytic Enzymes	--	--	--	--	--	--	--	--	--
12	Disinfectants	Protes disinfectant spray	Spray	Low	Parker Laboratories, Inc.	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
13	Disinfectants	Anios Excel Surface Premium	Liquid	Low	Laboratories Anios	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
14	Disinfectants	Clinelint Universal disinfecting wipes	Wipe	Low	GAMA Healthcare Ltd	Quaternary Ammonium(N-Alkyl)	●	●	●	--	●	●	●	●	●
15	Disinfectants	Asepti-Wipes	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
16	Disinfectants	Asepti-Wipes II	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
17	Disinfectants	Bleach 5.25%	Liquid	Low	All manufacture	Sodium Hypochlorite	--	--	--	--	--	--	--	--	--
18	Disinfectants	CarWipes	Wipe	Low	Metrox	Alcohol	--	--	--	--	--	--	--	--	--
19	Disinfectants	Cleansept Wipe forte	Wipe	Low	Dr Schumacher GmbH	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
20	Disinfectants	Cleansept Wipes	Wipe	Low	Dr Schumacher GmbH	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
21	Disinfectants	Gigasept AF	Liquid	Low	Schulke & Mayr GmbH	Quaternary Ammonium(N-Alkyl)	--	--	--	●(30)	●(30)	--	●(30)	--	--
22	Disinfectants	Incidin Foam	Spray	Low	Ecolab	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
23	Disinfectants	Metrox Wipes	Wipe	Low	Metrox	Alcohol	--	--	--	--	--	--	--	--	--
24	Disinfectants	Milton anti bacterial solution	Liquid	Low	Milton	Sodium Hypochlorite	--	●	●	--	--	--	--	--	--
25	Disinfectants	Nuclean	Liquid	Low	Alkapharm	Alcohol	--	--	--	--	--	--	--	--	--
26	Disinfectants	Sani-Cloth Active	Wipe	Low	PGI	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
27	Disinfectants	Sani-Cloth AF-Germicidal	Wipe	Low	PGI	Quaternary Ammonium(N-Alkyl)	--	●	●	--	--	--	--	--	--
28	Disinfectants	Sani-Cloth AF-Germicidal	Wipe	Low	PGI	Quaternary Ammonium(N-Alkyl)	●	●	●	--	--	--	--	--	--
29	Disinfectants	Sani-Cloth Bleach Germicidal	Wipe	Low	PGI	Sodium Hypochlorite	●	●	●	--	--	--	--	--	--
30	Cleaner	Endozime	Liquid	Cleaner	The Ruhoff Corporation	Proteolytic Enzymes	--	--	--	--	--	--	--	--	--
31	Disinfectants	Sani-Cloth HB	Wipe	Low	PGI	Quaternary Ammonium(N-Alkyl)	--	●	●	●(30)	●(30)	--	●(30)	●	--
32	Disinfectants	Sani-Cloth Plus	Wipe	Low	PGI	Quaternary Ammonium(N-Alkyl)	--	●	●	●(30)	●(30)	--	●(30)	●	--
33	Disinfectants	Cide Wipes	Wipe	Low	MEDICALS	Quaternary Ammonium(N-Alkyl)	--	●	●	--	--	--	--	--	--
34	Disinfectants	Sonowipes	Wipe	Low	Advanced Ultrasound Solutions Inc.	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
35	Disinfectants	Ster-Bar-Blu	Liquid	Low	Ecolab	Quaternary Ammonium(N-Alkyl)	--	--	--	●(50)	●(50)	--	●(50)	--	--
36	Disinfectants	Super Sani-cloth	Wipe	Low	Ecolab	Quaternary Ammonium(N-Alkyl)	--	--	--	●(15)	●(15)	--	●(15)	--	--
37	Cleaner	Transsept spray	Spray	Cleaner	Parker Laboratories, Inc.	Alcohol	--	●	●	●(15)	●(15)	--	●(15)	--	--
38	Disinfectants	T-Spray	Spray	Low	Pharmaceutical Innovations, Inc.	Quaternary Ammonium(N-Alkyl)	●	●	●	●(20)	●(20)	--	●(20)	--	--
39	Disinfectants	T-SprayII	Spray	Low	Pharmaceutical Innovations, Inc.	Quaternary Ammonium(N-Alkyl)	●	●	●	●(20)	●(20)	--	●(20)	--	--
40	Disinfectants	Aniozyme 1000	Liquid	High	Laboratories Anios	Peracetic Acid	--	--	--	--	--	--	--	--	--
41	Disinfectants	Cide Plus	Liquid	High	Advanced Sterilization Products	Glutaraldehyde	●	●	●	--	--	●(50)	--	--	--
42	Disinfectants	Cidex OPA	Liquid	High	Advanced Sterilization Products	Ortho-phthalaldehyde	●	●	●	●(15)	●(15)	--	●(15)	●	●
43	Disinfectants	Cidex Plus	Liquid	High	Advanced Sterilization Products	Glutaraldehyde	●	●	●	●(25)	●(25)	--	●(25)	●	●
44	Disinfectants	Gigasept	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde, formaldehyde	--	--	--	--	--	--	--	--	--
45	Disinfectants	Gigasept FF	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde	--	--	--	●(30)	●(30)	--	●(30)	--	--
46	Disinfectants	Hibiclens	Liquid	High	Regis Medical	Chlorhexidine gluconate	--	●	●	●(30)	●(30)	--	●(30)	--	--
47	Disinfectants	Metricide 28	Liquid	High	Metrox	Glutaraldehyde	●	●	●	--	--	--	--	●(50)	●
48	Disinfectants	Metricide 30	Liquid	High	Metrox	Glutaraldehyde	●	●	●	--	--	--	--	●(50)	●
49	Disinfectants	Metricide 30	Liquid	High	Metrox	Glutaraldehyde	●	●	●	--	--	--	--	●(50)	●
50	Disinfectants	Metricide OPA Plus	Liquid	High	Metrox	Ortho-phthalaldehyde	--	--	--	--	--	--	--	--	--
51	Disinfectants	Omicide- FQ2	Liquid	High	Coventry Chemicals Limited	Glutaraldehyde	--	--	--	--	--	--	--	--	--
52	Disinfectants	Omicide-14	Liquid	High	METREX RESEARCH CORPORATION	Glutaraldehyde	--	●	●	--	--	--	--	--	--
53	Disinfectants	Omicide-28	Liquid	High	METREX RESEARCH CORPORATION	Glutaraldehyde	--	●	●	--	--	--	--	--	●
54	Disinfectants/Sterilizer	PerASafe™	Powder	High	Dupont	Citric Acid	--	--	--	●(15)	●(15)	--	●(15)	--	●
55	Disinfectants	Salvanios pH 7	Liquid	High	Laboratories Anios	Glutaraldehyde	--	--	--	--	--	--	--	--	--
56	Disinfectants	Salvanios pH10	Liquid	High	Laboratories Anios	Glutaraldehyde	--	●	●	--	--	--	--	--	--
57	Disinfectants	Sekusept Extra	Liquid	High	Ecolab	Glutaraldehyde	--	--	--	--	--	--	--	--	--
58	Disinfectants	Sekusept Plus	Liquid	Low	Ecolab	Glucoprotamin	--	--	--	--	--	--	--	--	--
59	Disinfectants	Sporox	Liquid	High	DSHealthcare Inc.	Hydrogen Peroxide	●	●	●	--	--	--	--	●	●
60	Disinfectants	Sporox II	Liquid	High	DSHealthcare Inc.	Hydrogen Peroxide	●	●	●	--	--	--	--	●	●
61	Disinfectants	Steranios 2%	Liquid	High	Laboratories Anios	Glutaraldehyde	--	●	●	--	--	--	--	--	--
62	Disinfectants	REVITALiDx RESERT	Liquid	High	Steris	Hydrogen Peroxide	--	--	--	--	--	--	--	--	--
63	Disinfectants	Surfaces Hautes	Spray	High	Laboratories Anios	Glutaraldehyde	--	--	--	--	--	--	--	●	●
64	Disinfectants	Tristel Duo	Foam	High	Tristel Solutions Limited	Chlorine dioxide	●	●	●	--	--	--	--	--	--
65	Disinfectants	Tristel Sportoidal Wipe	Wipe	High	Tristel Solutions Limited	Chlorine dioxide	●	●	●	--	--	--	--	--	--
66	Disinfectants	Virkon	Powder	High	Dupont	Potassium peroxymonosulfate	--	--	--	--	--	--	--	--	--
67	Disinfectants	Wavicide-01	Liquid	High	National Therapy Products Inc.	Glutaraldehyde	--	●	●	●(50)	●(50)	--	●(50)	●	--
68	Disinfectants	Gigasept PA4	Liquid	High	Schulke & Mayr GmbH	Hydrogen peroxide, Acetic Acid, Peracetic Acid	--	●	●	●(15)	●(15)	--	●(15)	●	●
69	Disinfectant System	Ambaspa S1	Liquid	High	germics	UV-C	--	●	●	--	--	--	--	--	●
70	Disinfection System	Trophon	Vapor	High	Nanosonics Limited	Hydrogen Peroxide	●	●	●	--	--	--	--	--	--
71	Disinfection System	Trophon2	Vapor	High	Nanosonics Limited	Hydrogen Peroxide	●	●	●	--	--	--	--	--	--
72	Gel	Aquasonics 100	Gel	Gel	Parker Laboratories, Inc.	NA	●	●	●	●	●	--	●	●	●
73	Gel	Clear Image	Gel	Gel	NEC Medical Products	NA	●	●	●	●	●	--	●	●	●
74	Gel	Natural Image	Gel	Gel	Sonotech Inc.	NA	●	●	●	●	●	--	●	●	●
75	Gel	Scan	Gel	Gel	Parker Laboratories, Inc.	NA	●	●	●	●	●	--	●	●	●
76	Gel	Sonogel	Gel	Gel	Sonogel Vertriebe GmbH	NA	●	●	●	●	●	--	●	●	●
77	Gel	WaveLength	Gel	Gel	National Therapy Products Inc.	NA	●	●	●	●	●	--	●	●	●
78	Disinfectants	Acrylan	Liquid	Low	Antiseptica Chem. Phar. Produkte	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
79	Disinfectants	SEPTOL Linettes	Wipe	Low	ACTEON	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
80	Disinfectants	Wip Anios Excel	Wipe	Low	Laboratories Anios	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
81	Cleaner	EmpPower	Liquid	Cleaner	Metrox	Propylene glycol	--	--	--	--	--	--	--	--	--
82	Disinfectants	Hexanios GR	Liquid	Low	Laboratories Anios	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
83	Disinfectants	Oxivir TB Wipe	Wipe	High	Diversey	Hydrogen Peroxide	--	--	--	--	--	--	--	--	--
84	Disinfectants	Bacilcol 30 Foam	Spray	Low	BODE Chemie GmbH (HARTMANN)	Quaternary Ammonium(N-Alkyl), Alcohol	--	--	--	--	--	--	--	--	--
85	Disinfectants	Carvicide	Spray	Low	Metrox	Quaternary Ammonium(N-Alkyl), Alcohol	--	--	--	--	--	--	--	--	--
86	Disinfectants	Cingel Universal Spray	Spray	Low	GAMA Healthcare Ltd	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
87	Disinfectants	Gigasept FF neu	Liquid	High	Schulke & Mayr GmbH	Succinaldehyde	--	--	--	--	--	--	--	--	--
88	Disinfectants	Envirocide	Spray	Low	Metrox	Quaternary Ammonium(N-Alkyl), Alcohol	--	--	--	--	--	--	--	--	--
89	Disinfectants	MetricideGuard	Spray	Low	Metrox	Quaternary Ammonium(N-Alkyl), Alcohol	--	--	--	--	--	--	--	--	--
90	Disinfectants	Chlorox 100PPM	Liquid	High	All manufacture	Chlorine compound	--	--	--	--	--	--	--	--	--
91	Disinfectants	Chlorox 100	Liquid	High	All manufacture	Sodium percarbonate	--	--	--	--	--	--	--	--	--
92	Disinfectants	SCOPANIOS RC	Liquid	High	Laboratories Anios	Hydrogen peroxide, Peracetic Acid	--	--	--	--	--	--	--	--	--
93	Cleaner	EmpPower Foam	Liquid	Cleaner	Metrox	Polyethylene glycol	--	--	--	--	--	--	--	--	--
94	Cleaner	MetricLube	Liquid	Cleaner	Metrox	Propylene glycol	--	--	--	--	--	--	--	--	--
95	Cleaner	MetricShine	Liquid	Cleaner	Metrox	Phosphoric Acid, Glycolic Acid	--	--	--	--	--	--	--	--	--
96	Cleaner	MetricSponge	Liquid	Cleaner	Metrox	Propylene glycol	--	--	--	--	--	--	--	--	--
97	Sterilization system	V-PRO	Vapor	High	Steris	Hydrogen peroxide	--	--	--	--	--	--	--	--	--
98	Disinfectants	Chloroxidox Lösung	Liquid	High	vitalundfritm100 UG	Chlorine dioxide	--	--	--	--	--	--	--	--	--
99	Cleaner	Endozime extreme Power	Liquid	Cleaner	Ruhoff Healthcare	Proteolytic Enzymes	--	--	--	--	--	--	--	--	--
100	Disinfectants	Surfactos Premium	Liquid	Low	Anios	Quaternary Ammonium(N-Alkyl), Alcohol	--	--	--	--	--	--	--	--	--
101	Disinfectants	Virkon S	Liquid	High	Dupont	Potassium peroxymonosulfate	--	--	--	--	--	--	--	--	--
102	Disinfectants	V-Wipes	Wipe	Low	Whiteley Corporation	Alcohol	--	--	--	--	--	--	--	--	--
103	Disinfectants	terralin PAA	Liquid	High	Schulke	Hydrogen peroxide, Acetic Acid, Peracetic Acid	--	--	--	--	--	--	--	--	--
104	Disinfectants	Ricide/SPR/28	Liquid	High	Carth Medical	Ortho-phthalaldehyde	--	--	--	--	--	--	--	--	--
105	Cleaner	Tristel Pre-clean wipe	Wipe	Cleaner	Tristel Solutions Limited	Alcohol	--	--	--	--	--	--	--	--	--
106	Disinfectants	Biquazid S	Liquid	Low	Antiseptica Chem	Quaternary Ammonium(N-Alkyl), Alcohol	--	--	--	--	--	--	--	--	--
107	Disinfectants	deconex11 UNIVERSAL	Liquid	High	Borer Chemie AG	Potassium Pyrophosphate, Potassium hydroxide,	--	--	--	--	--	--	--	--	--
108	Disinfectants	MEDI CLEN	Liquid	Low	UBM Cleaning Solution INC	Sodium hypochlorite	--	--	--	--	--	--	--	--	--
109	Disinfectants	saFacide	Wipe	Low	Liebecos	Quaternary Ammonium(N-Alkyl)	--	--	--	--	--	--	--	--	--
110	Disinfectants	SURSA SAFE	Liquid	Low	Anios	Quaternary Ammonium(N-Alkyl)	--</								

Validated High Level Disinfection Instruction for Samsung Endo Cavity transducers


✚ Tristel Duo – High Level Disinfection Method

1. Personal Protection

- ▶ Personal protection must always be worn when handling possibly contaminated ultrasound probes. Personal protection at least includes gloves and eye protection.
- ▶ Once you are wearing personal protection, you are ready to begin the cleaning and disinfection process.

2. Cleaning the Probe

- ▶ The first step in the high-level disinfection process is thorough cleaning. Possibly contaminated probes must be thoroughly cleaned before they can be disinfected.
- ▶ Hold the probe and put one aliquot of Tristel duo in the indentation of the probe ❶
- ▶ Use a swap to spread the Tristel Duo liquid in the indentation(s) ❷
- ▶ Put 2 aliquots of Tristel Duo on a Tristel Dry Wipe and clean the transducer from handgrip to the tip of the probe ❸
- ▶ Use another dry swap to dry and clean the indentation(s) ❹

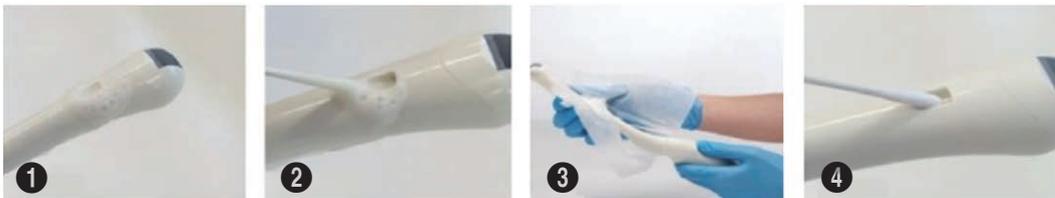

3. Disinfecting the Probe

- ▶ Tristel Duo incorporates two separate compartments that contain the Tristel Base and the Tristel Activator solutions that create chlorine dioxide when mixed (1:1). Tristel Duo foam is generated by depressing the incorporated pump.
- ▶ Change your gloves.
- ▶ Hold the probe and put one aliquot of Tristel duo in the indentation of the probe.
- ▶ Use a swap to spread the Tristel Duo liquid in the indentation(s).
- ▶ Put 2 aliquots of Tristel Duo on a Tristel Dry Wipe and disinfect the transducer with rotations from handgrip to the tip of the probe. Leave the disinfectant on the probe for at least 30 seconds.
- ▶ Use another dry swap to dry the indentation(s).
- ▶ Leave the surface to dry to ensure a 30 second contact time.



- ▶ Discard the used wipes, swaps and gloves to clinical waste, do not macerate. Do not re-use.
- ▶ Disinfected probe should be used immediately or stored in a manner to minimize recontamination.

❏ CIDEX® OPA Solution – High Level Disinfection Method

1. Personal Protective Equipment

- ▶ Personal protective equipment must always be worn when handling contaminated instruments and equipment. Personal protective equipment includes gloves, eye protection and fluid-repellent gown.
- ▶ Once you are wearing personal protective equipment, you are ready to begin the disinfection process.

2. Clean Instruments

- ▶ The first step in the high-level disinfection process is thorough cleaning. Contaminated probes must be thoroughly cleaned with a detergent prior to disinfection, especially take care of ridges and biopsy guide mountings.
- ▶ Following cleaning, rinse probe surface with large amounts of fresh water to remove residual detergent.

3. Using CIDEX® OPA Solution


- ▶ Before using the solution, be sure to read the directions for use on the bottle label and package insert. The shelf life of an unopened bottle of CIDEX® OPA Solution is two years. The solution requires NO activation.

4. Test

- ▶ Concentration of this product during its reuse life must be verified by the CIDEX® OPA Test Strip prior to each use to determine that the concentration of ortho-phthalaldehyde is above the MEC of 0.3%.
- ▶ CIDEX® OPA Solution shall not be used beyond 14 days.

5. Disinfect

- ▶ Prepare a ready-to-use CIDEX® OPA (Ortho-phthalaldehyde) solution with a concentration of at least 0.55% using the manufacturer's instructions.
- ▶ Fill tray, basin or tall container with Cidex OPA at ambient room temperature (approximately 20–25°C) to a level allowing immersion of the probe up to the immersion line as shown below.

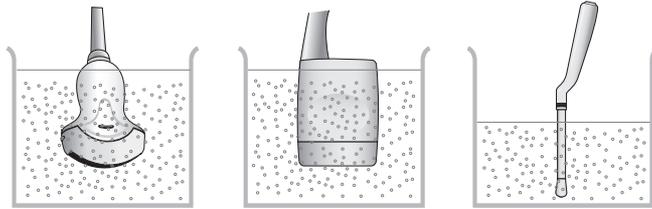

- ▶ Cover the CIDEX® Solution Tray with a secure lid. Soak probe for 12 minutes at 20–25°C to achieve high-level disinfection.
- ▶ Immerse probes in Cidex OPA at least until the immersion limits and ensure no air or bubbles are trapped. Allow soaking for minimum twelve (12) minutes.
- ▶ Excessive soaking time is not recommended as it may result in residues that result in patient allergic reaction or tissue discoloration.
- ▶ Cidex OPA solution shall not be reused beyond 14 days. Cidex OPA solution shall not be reused if concentration of active ingredient is below Minimum Effective Concentration of ortho-phthalaldehyde (0.3% w/v).

6. Rinse

- ▶ Remove probes from the solution and thoroughly rinse all surfaces up to the immersion line in shown in three (3) times with fresh sterile water, each time for a minimum of one (1) minute.

7. Dry

- ▶ Thoroughly dry all surfaces of the probe using a sterile, lint-free wipe or cloth, changing wipes/cloths when necessary to ensure the probe is completely dry. Visually inspect the probe to ensure all surfaces are clean and dry. Repeating drying steps if any moisture is visible.
- ▶ Disinfected probe should be used immediately or stored in a manner to minimize recontamination.

8. Dispose

- ▶ CIDEX® OPA Solution can be discarded down hospital and office drains or in accordance with local regulations.

❏ Trophon – High Level Disinfection Method

1. a Preparing the Probe

- ▶ Wearing gloves, clean and dry the probe BEFORE commencing the high level disinfection process as per the probe manufacturer's instructions.

2. b Clean Instruments

- ▶ The first step in the high-level disinfection process is thorough cleaning. Contaminated probes must be thoroughly cleaned with a detergent prior to disinfection, especially take care of ridges and biopsy guide mountings.
- ▶ Following cleaning, rinse probe surface with large amounts of fresh water to remove residual detergent.

3. Loading The Disinfectant Cartridge


- ▶ A Trophon disinfectant cartridge needs to be inserted into the Trophon EPR before a high level disinfection cycle can commence. Refer to the Trophon user manual for detailed instructions on how to insert and/or replace Trophon disinfectant cartridges.

4. Inserting the Chemical Indicator

- ▶ A Trophon Chemical Indicator must be used for each disinfection cycle. No other chemical indicators are approved for use in the trophon EPR. Each Chemical Indicator may only be used once. Place the Trophon Chemical Indicator into the holder. Refer to the Trophon Chemical Indicator IFU. You may then position the probe in the chamber.

5. Positioning the Probe

- ▶ When the Trophon is ready to use, screen message Load Probe and Indicator will be displayed.
- ▶ Open chamber door.
- ▶ Ensure the probe is correctly positioned in the chamber. The probe must not contact the chamber wall and must be positioned above the embossed line at the bottom of the chamber.

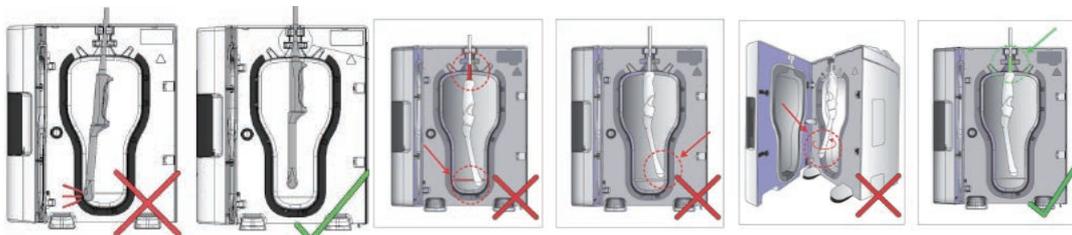

6. Closing the Chamber Door

- ▶ The door will automatically lock at the start of a high level disinfection cycle.

7. Disinfecting the Probe

- ▶ If the probe has been pre-cleaned and dried, press Yes using the soft key button. If Yes is selected, you will be prompted by the screen message: Press start to begin.
- ▶ The progress of the disinfection cycle is indicated on the LCD screen message: Disinfecting.
- ▶ The High Level Disinfection cycle will take 7 minutes to complete.

8. Removing the Probe

- ▶ NOTE: After completion of a successful high level disinfection cycle, the ultrasound probe and chamber may have surface temperatures up to 45°C and 60°C respectively. Care should be taken not to touch the chamber. The probe will feel warm to touch and is safe for handling and use with clean gloves on.
- ▶ Cycle Complete. Wear Gloves. Remove and Wipe Probe.
- ▶ Attention: Wear Gloves and Wipe Probe. (This message indicates that some hydrogen peroxide may not have been broken down into its constituents, oxygen and water and extra care should be taken when removing the probe).
- ▶ Wearing clean gloves, immediately remove the used Chemical Indicator from the trophon and verify the color change against the chart on the Chemical Indicator carton.
- ▶ Discard used Chemical Indicator.
- ▶ Remove the probe carefully using minimal contact after the cycle is complete. Avoid touching the probe against the chamber's hot surface. Avoid compromising the probe's disinfected surface before use.
- ▶ Wipe the probe with a clean, low lint, absorbent, single-use, dry sterile cloth/ wipe. Visually inspect the probe and ensure Any disinfectant residue present is removed.
- ▶ If a pass was verified by the Chemical Indicator color AND the trophon screen displayed cycle complete, the HLD has been successful.
- ▶ Discard gloves.
- ▶ Close the chamber door.
- ▶ Disinfected probe should be used immediately or stored in a manner to minimize recontamination.

*Note: Please refer to disinfectant manufacturer's user manual to find additional informations.

Gigasept PAA Concentrate – High Level Disinfection Method for TEE probe

1. Personal Protection

- ▶ Personal protective equipment must always be worn when handling possibly contaminated probes. Personal protective equipment includes gloves, eye protection and fluid-repellent gown.
- ▶ Once you are wearing personal protective equipment, you are ready to begin the disinfection process.

2. Cleaning the TEE Probe


- ▶ The first step in the high-level disinfection process is thorough cleaning. Possibly contaminated probes must be thoroughly cleaned prior to disinfection.
- ▶ Thoroughly clean the probe with a detergent especially taking care of the junctions of the flexible part and the indentation on the scan head (see Figure 1).
- ▶ Wipe the TEE probe with 5 Tristel Dry Wipes soaked with 8 ml 1.6% CIDEZYME and wipe for 3 minutes.
- ▶ Rinse the TEE probe for 1 minute with a 1% sodium dodecyl sulphate (SDS) solution.



[Figure 1. TEE probe MMPT3-7 with contamination spots (arrows)]

3. Disinfecting the TEE Probe

- ▶ Prepare a Gigasept® PAA solution with a concentration of at least 2% using the manufacturer's instructions.
- ▶ Place the TEE probe in the 2% Gigasept® PAA solution as shown in Figure 2.
- ▶ Ensure that all air is removed from the surface of the TEE probe.
- ▶ Leave the TEE probe for at 15 minutes in the solution.

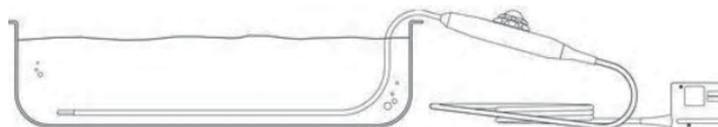

[Figure 2. Do not immerse beyond this point, 5cm from the strain relief]

- ▶ Gigasept® PAA solution shall not be reused beyond 12 hours.

4. Rinse

- ▶ Remove the TEE probe from the solution and thoroughly rinse all surfaces up to the immersion point as shown, repeat this process two more times with sterile water, each time for a minimum of one (1) minute.

5. Dry

- ▶ Thoroughly dry all surfaces of the probe using a sterile, lint-free wipe or cloth, changing wipes/cloths when necessary to ensure the probe is completely dry. Visually inspect the probe to ensure all surfaces are clean and dry. Repeat the drying steps if any moisture is still visible.
- ▶ Disinfected probe should be used immediately or stored in such a way that recontamination is prevented.

6. Disposal of the Disinfection Solution

- ▶ According to the manufacturer Gigasept® PAA Solution can be discarded down hospital- and office drains. Recommendation however is to check the local regulations first.